



# Inservice Guide - Point of Care

1

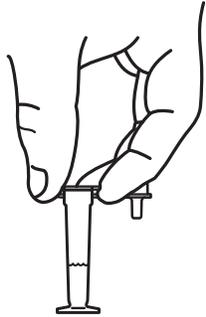


Remove contents from package. Holding the buffer vial in an upright position, remove the shipping cap as shown and set the vial aside.

2

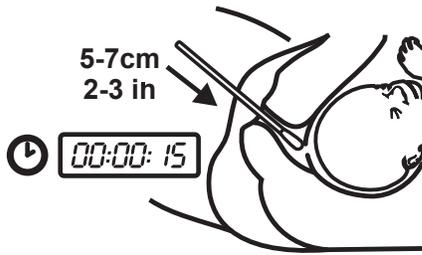


Remove swab from its sterile package and collect a sample from the vagina.

3

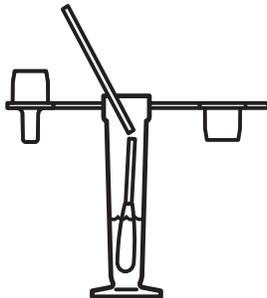


Place the swab in the buffer. Break off the swab tip at the scored mark and leave the tip in the vial.

4

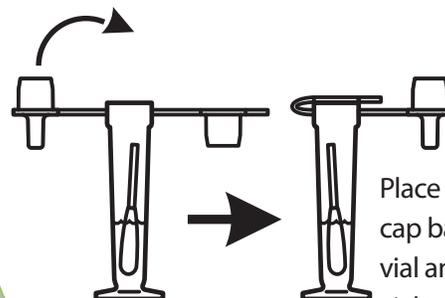


Place the shipping cap back on the vial and return the vial to the lab.

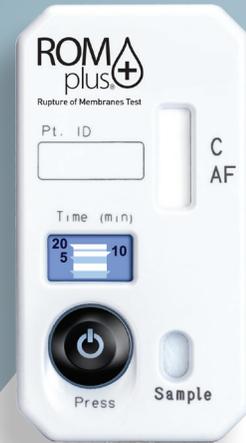

Warning: The test may report positive results in patients with intact membranes and therefore decisions to induce labor should not be based solely on the ROM Plus test results.


# Inservice Guide-Lab

1

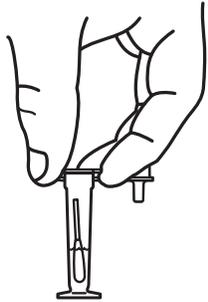


Holding the buffer vial in an upright position, remove the shipping cap as shown.

2

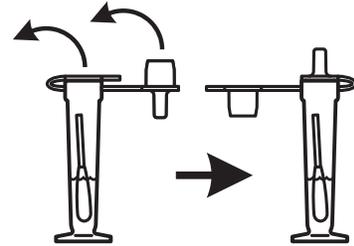

Replace shipping cap with drop dispenser cap.


3

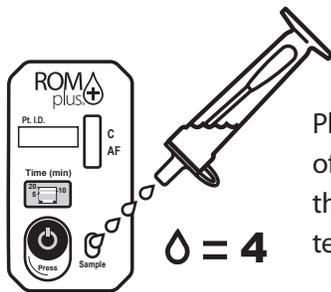

Place 4 to 6 drops of solution into the well on the test cassette.


4 = 4

4

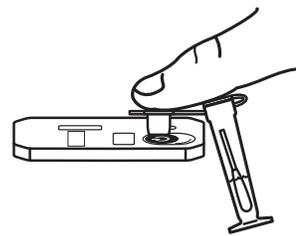


Place test cassette on a flat surface and start timer by pressing with your thumb from the bottom of the timer up.

5

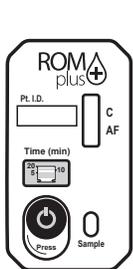

Wait 5-20 minutes for test result to manifest.

6

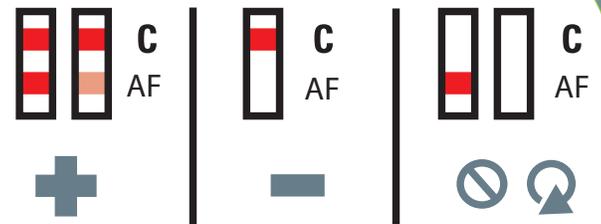

Read the test result.