


The **EXTRAORDINARY**
CARRIERE MOTION CLEAR
Class II Appliance

And **CLEAR** may be the least
extraordinary thing about it.

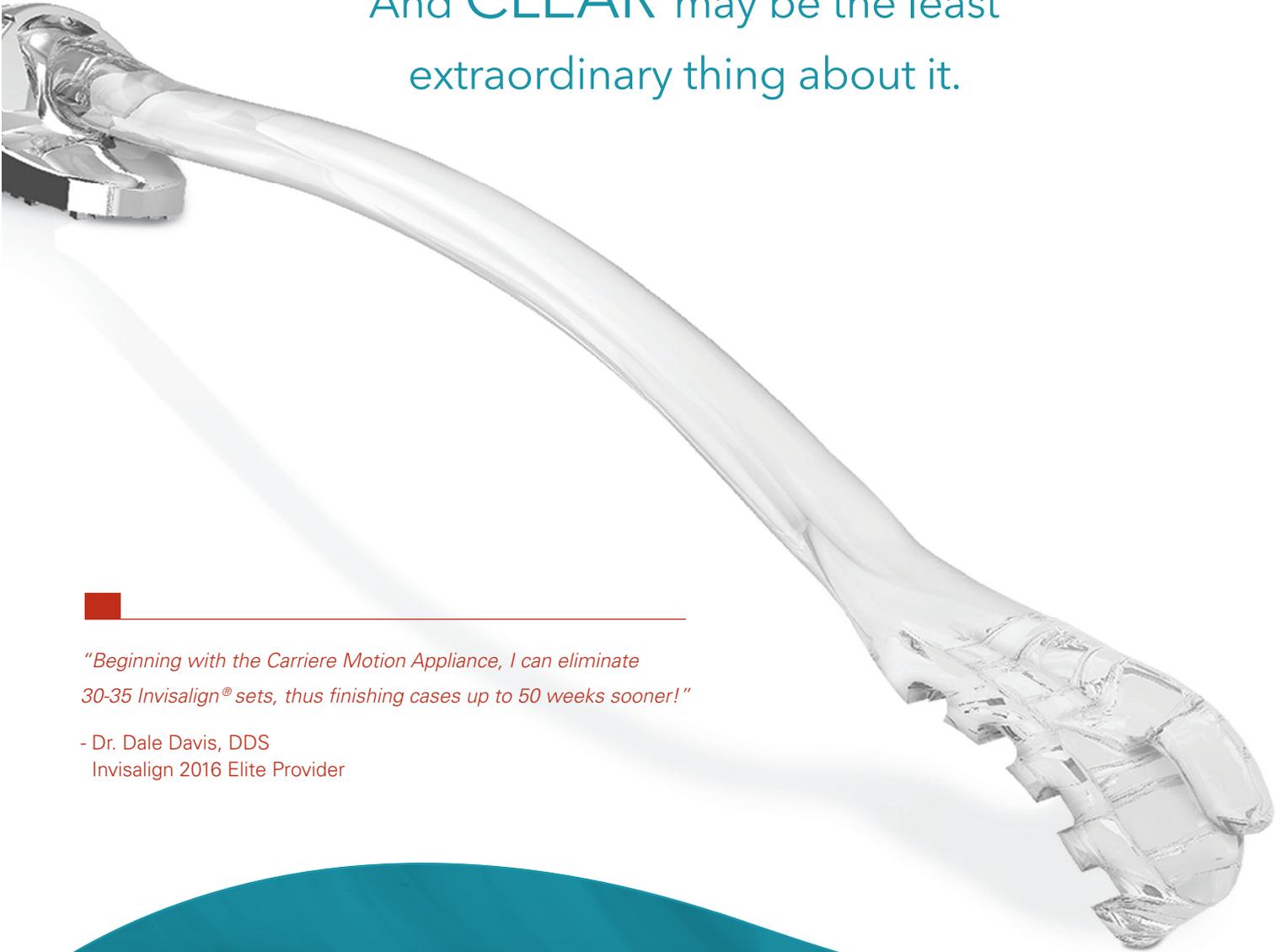

*"Beginning with the Carriere Motion Appliance, I can eliminate
30-35 Invisalign® sets, thus finishing cases up to 50 weeks sooner!"*

- Dr. Dale Davis, DDS
Invisalign 2016 Elite Provider

CARRIERE®

fast ■ natural ■ gentle



Introducing the **CARRIERE MOTION CLEAR** Class II appliance.

Like its metal counterpart, The *Carriere*® *Motion*™ Class II appliance is based on the philosophy of achieving a stable Class I platform in the posterior segment at the onset of treatment in order to simplify treatment and reduce patient time in braces or aligners. This new treatment paradigm eliminates competing force vectors and patient compliance issues inherent in traditional Class II correction.

Whether it's the simplicity-of-use, the sleek patient-friendly design, or the remarkable speed and predictability of how it works, the *Carriere Motion Clear*™ Class II Appliance is truly extraordinary.


Articulating ball and socket
for controlled molar rotation and uprighting

Smooth, low-profile, rounded arm
for patient comfort

Advanced medical grade polymer
provides excellent strength and durability


Contoured stainless steel base
to fit patient tooth anatomy

Simple, direct-bond
surface pads

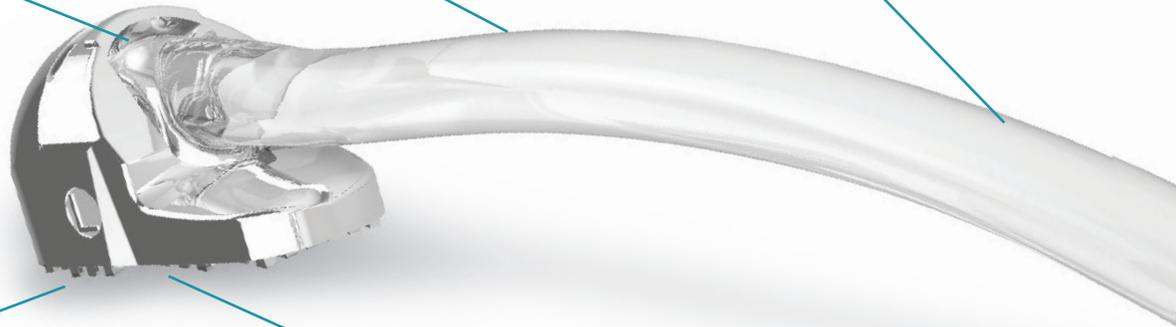

"The Carriere Motion Clear Appliance has proven to be perfect for my patients with high aesthetic demands. The Clear Motion has allowed me to significantly reduce the number of clear aligner trays per patient, almost in half, plus it maintains its clarity throughout treatment. I couldn't be happier... I have faster results, lower overall costs, and a Class II solution for all patients."

- Dr. Francesco Garino

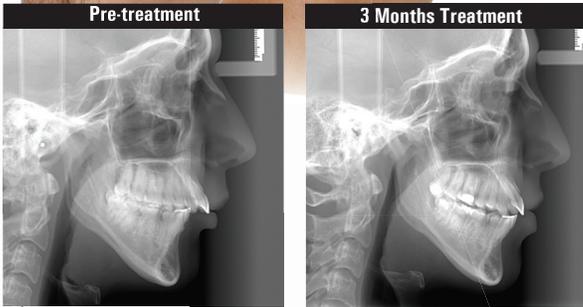

Initial



4 Weeks



8 Weeks



12 Weeks - Class I Occlusion Achieved



4 Months



Clear Aligner Efficiency




The *Carriere Motion Clear* Class II Appliance offers the ultimate in aesthetic treatment, quickly establishing a Class I platform with even the most difficult or complex Class II cases.

Exceptional aesthetics
guaranteed not to stain or discolor

Sleek hooks on pad
for attachment of the
Carriere Oral Elastics

Patented anterior pad with special dovetailed grooves
for excellent bond strength and retention

Fixed cuspid pad
allows the distal movement of the cuspid
along the alveolar ridge without tipping

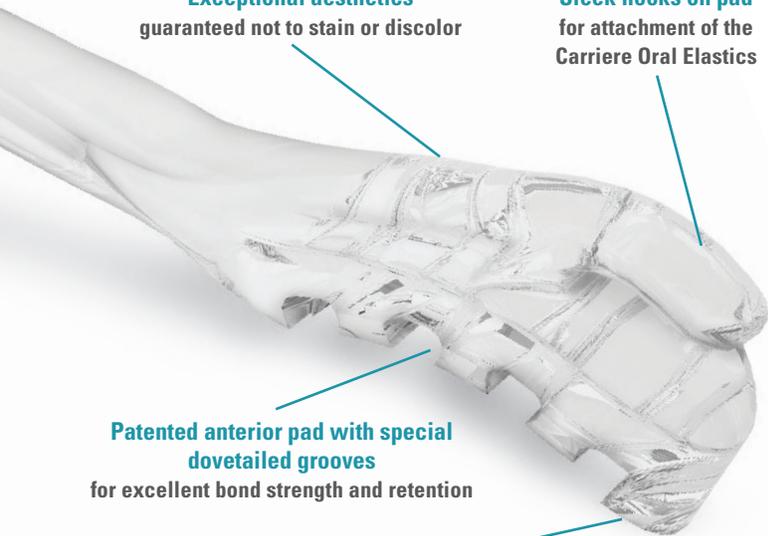

Description/Sizes	Color	Left	Right	Set
<i>Carriere Motion Clear</i> Class II 16 mm	○	424-816LC	424-816RC	424-816C
<i>Carriere Motion Clear</i> Class II 18 mm	●	424-818LC	424-818RC	424-818C
<i>Carriere Motion Clear</i> Class II 20 mm	●	424-820LC	424-820RC	424-820C
<i>Carriere Motion Clear</i> Class II 23 mm	●	424-823LC	424-823RC	424-823C
<i>Carriere Motion Clear</i> Class II 25 mm	●	424-825LC	424-825RC	424-825C
<i>Carriere Motion Clear</i> Class II 27 mm	●	424-827LC	424-827RC	424-827C
<i>Carriere Motion Clear</i> Class II Trial Kit (23, 25, 27 mm) 3-set				424-800C
<i>Carriere Motion Clear</i> Class II Intro Kit (16, 18, 20, 23, 25, 27 mm) (6 sets)				424-801C
<i>Carriere Motion Clear</i> Class II Standard Kit (20 sets)				424-802C
<i>Carriere</i> Typodont with <i>Motion Clear</i> Class II and Metal Class II with Aligner				631-017CMNE
<i>Carriere</i> Typodont with <i>Motion Clear</i> Class II with Clear Aligner				631-017CNE
<i>Carriere Motion Clear</i> Class II 5X Model				631-424CX
<i>Carriere Motion</i> Elastics-Force 1, 1/4", 6 oz. (50 packs of 100 elastics)				424-9F1
<i>Carriere Motion</i> Elastics-Force 2, 3/16", 8 oz. (50 packs of 100 elastics)				424-9F2



The New Movement In Orthodontics™

For more information on our products and educational offerings, please contact us:

HenryScheinOrtho.com

Outside the U.S.: **+(1) 760.448.8600**

Canada: CERUM **800.661.9567**

To email an order from outside the U.S.: **IntlOrders@OrthoOrganizers.com**

Ortho Organizers, Inc. 1822 Aston Ave. Carlsbad, CA 92008-7306 USA

© 2016 Ortho Organizers, Inc. All rights reserved. PN M1001 09/16