


## Instructions for use



Remove the cap and shake the inhaler. Hold it vertically and insert it into the base of the spacer device.

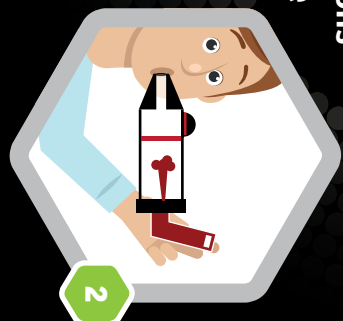

Squeeze the lips around the device's mouthpiece. Withdraw the device, wait for five to ten breaths (or place the mask over the nose and mouth if a mask is used). Administer a dose of spray. Breathe in and out normally four times through your mouth. The visual valve should move.



Withdraw the device, wait for five to ten breaths before the next puff and then start again, following your prescription. Remove the spray and replace the cap. Rinse the mouth.

**vision-air**  
- PLUS -

Spacer device  
+ 6 years and adult

6 years  
and +



175 ml



### Precautions for use

The spacer device must only be used under constant adult supervision, using medicines prescribed by a doctor. Consult a doctor immediately if the user's condition deteriorates.

A spacer device is for individual use only and should not be shared with other users. It must be replaced if the plastic becomes opaque, the wall is scratched or there is a crack that means it is no longer airtight. It must be replaced after 12 months' use in all cases. Keep out of reach of children.

### Maintenance/Cleaning

Clean the spacer once a week. Do not clean with boiling or hot water as this will damage your spacer. Remove the mask and base, wash all the components in soapy water and leave to dry. Do not rinse in clear water or wipe the components with a cloth. This will help retain its antistatic characteristics. Check information leaflet for more information.



PIP code: 1128423  
EAN code: 3401081796368  
Distributor  
TriOn Pharma UK  
Cowplain. PO8 8DL  
info@trionpharma.co.uk

Manufactured  
in France



**vision-air**  
- PLUS -

Spacer device  
+ 6 years and adult

Spacer  
device

+ 6 years & adult


6 years  
and +



- Visual valve
- Anti-microbial valve
- Universal base
- Reusable
- Antistatic
- BPA and latex-free
- Optional silicon mask

CE

175 ml

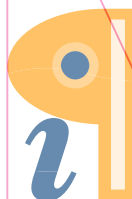

Edited with **Infix PDF Editor**  
- free for non-commercial use.

To remove this notice, visit:  
[www.iceni.com/unlock.htm](http://www.iceni.com/unlock.htm)