



Numero de Téléphone 0 811 38 60 80
Tél. 0 811 38 60 80
0 811 38 60 80

France
38330 Montbonnot-Saint-Martin
53 Route du Port
38570 Beauvois-en-Royans
stplastics@stplastics.com
www.stplastics.fr

Stplastics
Pharmalup Laboratoires
38570 Beauvois-en-Royans
53 Route du Port
38330 Montbonnot-Saint-Martin
stplastics@stplastics.com
www.stplastics.fr

+ 6 years and adult
Spacer device

visionair
-PLUS-

visionair
-PLUS-

Spacer
device
+ 6 years & adult



- Visual valve
- Anti-microbial valve
- Universal base
- Reusable
- Antistatic
- BPA and latex-free
- Optional silicon mask

175 ml
CE
Manufactured in France



Precautions for use

The spacer device must only be used under constant adult supervision, using medicines prescribed by a doctor. Consult a doctor immediately if the user's condition deteriorates.

A spacer device is for individual use only and should not be shared with other users. It must be replaced if the plastic becomes opaque, the wall is scratched or there is a crack that means it is no longer airtight. It must be replaced after 12 months' use in all cases. Keep out of reach of children.

Maintenance/Cleaning

The spacer device should be cleaned before it is used for the first time and then once a week. Remove the mask and base, wash all the components in soapy water and leave to dry. Do not rinse in clear water or wipe the components with a cloth. This will help retain its antistatic characteristics.



VisionAIR Plus spacer device
Children + 6 years and adults
LPPR code: 1128423
ACL code: 3401081796368

visionair
-PLUS-

Spacer device
+ 6 years and adult

6 years
and +



Remove the cap and shake the inhaler. Hold it vertically and insert it into the base of the spacer device.



1

Squeeze the lips of the device's mouthpiece (or place the mask over the nose and mouth if a mask is used). Administer a dose of spray. Breathe in and out normally four times through your mouth. The visual valve should move.



2

Withdraw the device, wait for five to ten breaths before the next puff and then start again, following your prescription. Remove the spray and replace the cap. Rinse the mouth.



3

Instructions
for use