


## Instructions for use



Remove the cap and shake the inhaler. Hold it vertically and insert it into the base of the spacer device. Place the mask over the child's nose and mouth to create a seal.

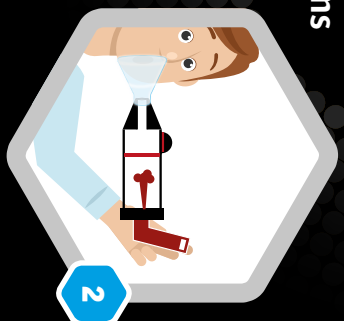

Administer a dose of spray. Breathe in and out normally four times through your mouth. The visual valve should move. Withdraw the device from the child's mouth, wait for five to ten breaths before the next puff and then start again, following their prescription.



Remove the spray and replace the cap. Give the child a drink and rinse their face.

visionair  
- PLUS -

Spacer device  
for babies and children  
(including mask)

0-6  
years



175 ml CE

## Precautions for use

The spacer device must only be used under constant adult supervision, using medicines prescribed by a doctor. Consult a doctor immediately if the user's condition deteriorates.

A spacer device is for individual use only and should not be shared with other users. It must be replaced if the plastic becomes opaque, the wall is scratched or there is a crack that means it is no longer airtight. It must be replaced after 12 months' use in all circumstances. Keep out of reach of children.

## Maintenance/Cleaning

The spacer device should be cleaned before it is used for the first time and then once a week. Remove the mask and base, wash everything in soapy water and leave to dry. Do not rinse in clear water or wipe the components with a cloth. This will help retain its antistatic characteristics.



VisionAIR Plus spacer device  
Baby/child 0-6 years  
LPPR code: 1134783  
ACL code: 3401081796139

Manufactured  
in France



Manufacturer

Stplastics  
38160 Beauvoisin-en-Royans  
53 Route du Port  
385 rue Lavoisier

Pharmap Laboratoires

Distributor

38330 Montbonnot-Saint-Martin  
FRANCE

sticare@stplastics.com  
www.visionairplus.fr

Toll-free  
number  
0 811 38 60 80  
e0.06 INCL. TAX/MIN



visionair  
- PLUS -

Spacer  
device  
for babies and children  
(including mask)

0-6  
years



- Flexible silicon mask
- Visual valve
- Anti-microbial valve
- Universal base
- Reusable
- Antistatic
- BPA and latex-free



CE  
175 ml

Spacer device  
for babies and children (including mask)